



Medical-D

REprogramming of D-mode, F-mode and SMILE-mode

In some situations, the programmed D-mode and SMILE-mode may be lost, for example when this camera is not used for longer time or when batteries are not charged timely. The steps below enable you to RE-programm the Dial Mode. KEEP THIS CARD WELL ARCHIVED. For regular functions on this camera see also the digital instruction manual of SONY;

You can download it via this link: http://rd1.sony.net/help/dsc/1815/h_zz

- 01 Controlwheel
- 02 Dial Mode
- 03 MENU-button
- 04 Fn (Function)-button
- 05 Shutter (+ Zoom)
- 06 OK / Enter button

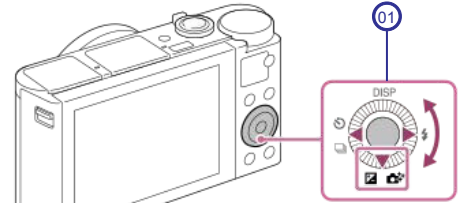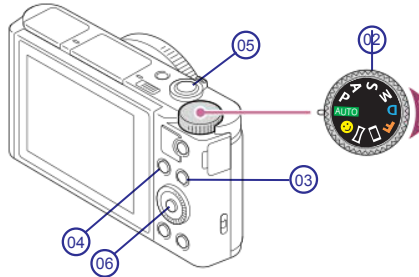

INITIALIZING

MENU > > Setup7 > Setting Reset > OK > Initialize > Enter > OK > Enter > OK
> Choose Language > Set Area / Date / Time

PROGRAMMING

- 1 Set the Dial Mode on 'M'
- 2 Push MENU-button > turn Controlwheel while changing functions and go to menu >

- 1/4 AF (Focus Area) > select > 'Center'
- 1/5 (Exposure) > ISO > select > 'ISO AUTO'
- (Exposure) > Metering Mode > select > 'Center'
- 1/6 (Flash) > go to > Flash Comp. > select > '±0.0'
- 1/7 (Color/WB/Img.Processing) > go to > White Balance > Flash
> select > 'A-B: B1.5'
- 2/5 (Display/Autoreview) > go to > Gridline >
select > 'Diag. + Square Grid'
- (Auto Review) > select > 2 sec

Press MENU-button to leave the menu

- 3 Go to Shutter > select Zoom > '100 mm'
- 4 Turn Controlwheel > select Shuttertime > '1/60'
- 5 Press 1 x 'down' on Controlwheel >
> Rotate aperture to > 'F8.0'
- 6 1/3 Go to Memory > 1 / 2 > select > 'Memory'
> select with Controlwheel > 1 > 'OK'
> leave the menu
- 7 1/7 (Color/WB/Img.Processing) > go to > White Balance > Flash
> select > 'A-B: B2.5'
- Press 1x 'down' on Controlwheel > > select > 'F6.3'
- 8 1/3 Go to Memory > 1 / 2 > select > 'Memory'
> Select with Controlwheel > 2 > 'OK'
> leave the menu

Medical-D is a registered Trademark

The Medical-D camera is designed & produced by DHM-dental B.V.

version: Oct 2020

file: TD2.2_ProgramCart dutch-english MK2)_nov-2020

MDR:2017/745

first class performance

DHM
DENTAL


DHM-dental B.V.
Simon Stevinweg 48
6827 BT ARNHEM
Netherlands

Medical-D

HERprogrammeren van D-mode, F-mode en SMILE-mode

Er zijn situaties waarbij de voorgeprogrammeerde D-mode en SMILE-mode verloren kan gaan, bijvoorbeeld wanneer deze camera langere tijd niet is gebruikt of de batterijen niet op tijd geladen worden. Met de hieronder vermelde stappen kunt u de Dial Mode (Functiekeuzeknop) HER-programmeren. **BEWAAR DEZE KAART GOED.** Voor de reguleren functies van deze camera raadpleegt u de digitale instructies van SONY; deze kunt u downloaden via de volgende link: http://rd1.sony.net/help/dsc/1815/h_zz

- 01 Besturingswiel
- 02 Functiekeuzeknop
- 03 MENU-knop
- 04 Fn (Functie)-knop
- 05 Ontspanknop (+ zoom)
- 06 OK / Enter knop

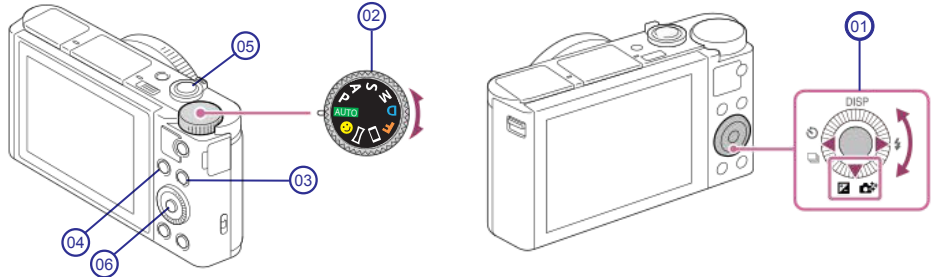






INITIALIZEREN

MENU >  > Instellingen7 > Instellingen herstellen > OK > Initialiseren > Enter > OK > Enter > OK > Stel taal in > Stel datum, plaats en tijd in

PROGRAMMEREN

- 1 Functiekeuzeknop op 'M' instellen
- 2 MENU-knop indrukken > draai met besturingswiel door functies en ga naar menu >

-  1/4 AF (Scherpstelgebied) > selecteer > 'midden'
-  1/5 (Belichting) > ISO > selecteer > 'ISO 200'
(Belichting) > Lichtmeting > selecteer > 'midden'
-  1/6 (Flitslicht) > ga naar > Flitscompensatie > Stel in op > '±0.0'
-  1/7 (KleurWB/beeldverwerking) > ga naar > Witbalans > Flash > selecteer > 'A-B : B1.5'
-  2/5 (Weergave/autom.weergave) > ga naar > Stramienlijn > selecteer > 'Diag. +Vierkantsr.'
(Automatische weergave) > selecteer > 2 sec

Druk op de MENU-knop om het menu te verlaten


- 3  Ga naar de Ontspanknop > stel Zoom in > '100 mm'
- 4  Draai aan Besturingswiel > Sluittijd instellen > '1/60'
- 5  Druk 1 x de onderkant van Besturingswiel in >   > draai diafragma > 'F8.0'
- 6  1/3 Ga naar Geheugeninstellingen >   1/  2 > selecteer > 'Geheugen' > selecteer met Besturingswiel > 1 > 'OK' > verlaat het menu
- 7  1/7 (KleurWB/beeldverwerking) > ga naar > Witbalans > Flash > selecteer > 'A-B : B2.5'
Press 1x 'down' on Controlwheel >   > select > 'F6.3'
- 8  1/3 Ga naar Geheugeninstellingen >   1/  2 > selecteer > 'Geheugen' > selecteer met Besturingswiel > 2 > 'OK' > verlaat het menu

Medical-D is a registered Trademark

The Medical-D camera is designed & produced by DHM-dental B.V.

version: Oct 2020

file: TD2.2_ProgramCart dutch-english MK2)_nov-2020

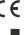MDR:2017/745 

first class performance



DHM-dental B.V.
Simon Stevinweg 48
6827 BT ARNHEM
Nederland