Quick Instruction Guide for sending images to a computer over a Wifi access point

This is a quick instruction for sending images to a computer connected to a wireless access point or a wireless broadband router using the inbuilt Wifi of the camera.

You can select a specific location on your computer to place copies of the images. In this way your preferred patient database program, for instance VisiQuick, can visualize the images and link the images directly to your patient.

Installation steps:

1 Installation of PlayMemories Home on your computer

Detailed information can be found on page 304 of the Sony User Guide

- Download PlayMemories Home at: <http://www.sony.net/pm/>
Install the program. Follow the instruction of the installation program. If asked: restart your computer.

On PC:

- Start PlayMemories Home
- Connect the camera to your computer using the micro USB cable (supplied), then turn the camera on.
- Alert: DSC-HX99 is connected. Do you want to display options for cameras? -> Choose Yes.
- Select: recommended. Choose: Next. Allow changes to be made.
- Choose: Complete.
- Go to Extra of PlayMemories Home -> Settings -> Import -> Folder for Import -> select: Import media to selected folder.
- Choose Wi-Fi import -> choose location for import
- Exit PlayMemories Home and disconnect Camera from your computer

On Mac:

- Start PlayMemories Home
- Connect the camera to your computer using the micro USB cable (supplied), then turn the camera on.
- Go to Settings of PlayMemories Home -> Import-> select: Import media to selected folder.
- Choose: automatic wireless import -> choose: 'setup' -> press 'OK'
- Choose the location for import -> press 'OK'
- Quit PlayMemories Home and disconnect Camera from your computer

2 Connect to your Wifi access point

Detailed information can be found on page 295 of the Sony User Guide

- Put the camera on.

On the camera go to: MENU -> NETWORK -> [Wi-Fi Settings] -> [Access Point Set]

Select the access point you want to register.

Input the password, and select [OK]

Now your camera is configured and ready to send images over WiFi to the computer.

3 Sending images to the computer

Detailed information can be found on page 287 of the Sony User Guide

- Start your computer
 - Start your favorite patient database program and select your patient
 - Put your camera on and take pictures of your patient.
 - Go on the camera to: MENU -> Network -> [Send to Computer].
- Now a copy of the images is placed on your computer and linked to your patient.
After the images are copied to your computer, the camera will shut off.

On Mac: after the import: choose on the screen: Import to DCS-HX99 completed: Close

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