



Cleaning and Disinfecting the Camera Handpiece

In a clinical use environment, the health care provider should wear protective disposable gloves and cover the camera with a hygienic barrier. Before using the camera the first time, and before every new patient, the following protocol is recommended:

1. Remove and discard all protective hygienic barriers and / or sheaths from the camera prior to removing disposable gloves.
2. Place the camera on a tray covered by a disposable liner, or in a receptacle that can be thoroughly disinfected.
3. Remove and discard gloves.
4. Wash hands and put on a new pair of disposable gloves.
5. Disconnect the camera from the USB cable.
6. If the camera or cable are visibly soiled (e.g., with blood or saliva), each should be cleaned with a cloth or paper towel, and then dried with a clean lint-free cloth or paper towel.
7. Thoroughly wipe the camera and cable (if applicable) with a disinfecting product (such as SONO Wipes) and wait 30 seconds. Do not expose the camera cable connector to any amount of liquid.
8. Repeat step 7. When the camera has been wiped two times, continue with the following steps.
9. Use a sterile dry lap sponge to dry the camera or cable, as needed.
10. Place the camera in a clean environment, ready for next use.
11. Reconnect the camera.
12. Remove and discard gloves.

Cleaning the Lens Surface

To clean the lens surface, moisten a soft cotton swab dipped in rubbing (isopropyl) alcohol. Gently wipe the lens surface end-to-end in straight lines without applying pressure; do not scrub. Allow the alcohol to evaporate from the lens surface. Repeat these steps, as needed, until the lens surface is clean.