



Finished in next to no time.



Saliva ejector and high volume suction tubings

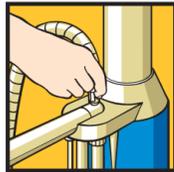


1. Aspirate water occasionally via the KaVo AQUAamat device

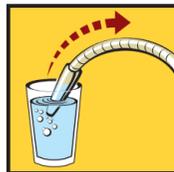

...if AQUAamat is not available, aspirate water occasionally from a suitable container or other clean water source



2. Place aspirator cannula into the measuring device, push down once and release

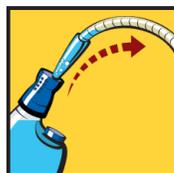

3. Aspirate the measured dose of DEKASEPTOL Gel from the measuring device; repeat steps 1-3



4. Remove the tubing and place back in its holder

Spittoon



1. Press down on the measuring cap



2. Pour a measured dose of DEKASEPTOL Gel into the spittoon



3. Distribute DEKASEPTOL Gel evenly using a brush and leave for approximately 5 minutes



4. Rinse and clean out the spittoon with the bowl flush again using a brush



Your dealer:

We can advise you competently, comprehensively and professionally. And it is our pleasure so to do. Information now also on the Internet: www.kavo.com



The Coconut-based disinfection and cleaning material.



KaVo. Dental Excellence.



KaVo. Dental Excellence.

Ready...

A clearing in the pathogen jungle.

Danger zone: suction unit. Dirt collects, bacterium, fungi and viruses thrive and are a threat to health. Hit germs with weapons of nature!

The main active ingredient of DEKASEPTOL is glucopeptamin – a coconut derivative. Glucopeptamin is an antimicrobial ingredient that far surpasses biguanide or phenol derivatives in strength and thoroughness. A clean, hygienic substance for suction devices, aspiration tubes and spittoons.

And for you, one reason to smile: what makes life hard for bacteria and viruses, makes your work really easy. But just have a look for yourselves!

Safe – clean – economical – quick:

- Highly effective: antibacterial, tuberculocidal, virucidal, fungicidal
- Economical: 6 litre refill container
- Environmentally friendly: free from aldehyde, phenol and chlorine
- Time-saving: thanks to the measuring pump, it is immediately active
- Practical: no mixing
- Registered on the DGHM listing

DEKASEPTOL works:

- To clean
- To deodorise
- To disinfect:

Antibacterial	✓	Tuberculocidal	✓
Fungicidal	✓	HBV effective	✓
Virucidal	✓	HIV effective	✓

First choice – best value.

Steady...

No stirring, shaking or mixing.

Spare yourself from time-consuming mixing and tedious application! DEKASEPTOL Gel is a ready-made blend and is immediately active.

Take care of yourself and your appliances. No mixing also means: no mistakes, no contact with sensitive hands, no dirty work surfaces, and no stress on materials. A completely clean affair.

Really economical. Due to the economical measuring cap, nothing can go wrong and nothing is wasted either. The best way to fill up is using the 6 litre refill container.

Disposal without worries. What comes from nature, also lives alongside it. DEKASEPTOL is completely environmentally harmless, and according to OECD guidelines, easily and quickly biodegradable.

Go!

An early finish means a long happy evening.

Gain time for yourself! DEKASEPTOL is not only easy to manage, it is also quick. Simply aspirate the gel into the tubing or pour into the spittoon. Leave to take effect. That's it.

Is it past 18:00 in the jungle? Not any longer! Because DEKASEPTOL will save you precious minutes on each suction unit.

"Peanuts" you say? "Large fat coconuts" we say. If you do your sums again, you will quickly arrive at a saving in time of 20 minutes per week.

That adds up to 2 whole working days a year. Time, which thanks to DEKASEPTOL, can be used in a better way. Earlier to leave, quicker home.

